

Red Light Therapy



CONTENTS

	Page
A: Introduction	3
1.1 Package Contents	
1.2 Features & Specifications	3
B: LED Safety Information	3
C: Battery Installation & Replacement	3
D: Usage Instructions	
1.1 Small Animals (Dog / Cat) Treatments	4
1.2 Medium Animals (Human) Treatments	4
1.3 Large Animals (Horse) Treatments	4
E: Product Support	4



Developed in Australia

A: Introduction

Thank you for purchasing the **WPT – Small Super RED Light**.
WPT wishes you every success with your new LED treatment Light.

1.1 Package Contents

Carefully unpack and remove the contents. If any of the following items are missing or damaged, contact Wozenphotonictherapy (WPT).

- **WPT Small Super RED LED Light** [3 x treatment settings]
- 1 x Protected 3.6V Rechargeable Battery 18650 (Li – ion)
- Horse Treatment Information Pack [Horse Kit only]
- Sturdy Storage Case [Horse Kit only]
- User Manual & Memory Stick
- Simple Health Maintenance Book (Optional)

1.2 Features & Specifications

The **WPT Small Super RED LED Light** has three power settings.

- Low [100mW = 0.1 Watt] for Small Animals
- Medium [500mW = 0.5 Watt] for Medium Animals
- High [1000mW = 1.0 Watt] for Large Animals

The Light Emitting Diode (LED) outputs a 660nm Deep Red Light.
Beam Angle of 120° from the LED surface.

Photonic Output is 1.09×10^{18} photons / second / cm at 1000mW.
Five year (5 YR) warranty for workmanship and product.

B: LED Safety Information

The **WPT – Small Super RED LED Light** is a **Class 1 / Low Risk LED** (battery – operated) Torch.

WARNING – Do not look directly at or shine light beam at eyes as this may cause temporary visual impairment.

The Small Super RED LED Torch should be used by **ADULTS ONLY**.

C: Battery Installation & Replacement

To install / replace / recharge the 3.6V battery provided simply unscrew Torch base to insert and replace to seal the Battery. **REMOVE** battery from LED Torch if not used for prolonged time period to avoid corrosion.